VitalSim™ Blood Pressure Trainer

ENGLISH
ESPAÑOL
ITALIANO
FRANÇAIS
DEUTSCH
NEDERLANDS
NORSK
SVENSKA
SUOMI
CHINESE
JAPANESE
KOREAN

Directions for Use
Instrucciones de Utilización
Istruzioni per l'Uso
Mode d'Emploi
Gebrauchsanweisung
Handleiding
Bruksanvisning
Bruksanvisning
Käyttöohje
使用说明书
取扱説明書
사용설명서

Norway: LAERDAL MEDICAL AS
PO. Box 377, N-4002 Stavanger
Tel. +47 51 51 17 00, Fax +47 51 52 35 57
E-mail: laerdal.norway@laerdal.no

Australia: LAERDAL Pty. Ltd. (ABN 47 003 817 490)
23 Edward Street, Oakleigh, Victoria 3166
Tel. +61 3 9569 4055, Toll free 1800 331 565
Fax +61 3 9569 4028, Toll free 1800 635 835
E-mail: customerservice@laerdal.com.au

Benelux: "The Laerdal BeNeLux House"
Dijkstraat 54, NL-5554 PS VALKENSWAARD
Tel. +31 40 208 58 00, Fax +31 40 208 58 02
E-mail: info@laerdal.nl

Benelux: Belgium and Luxembourg
LAERDAL BeNeLux nv
Mechelsesteenweg 277, B-1800 VILVOORDE
Tel. +32 2 253 36 96, Fax + 32 2 253 36 90
E-mail: info@laerdal.be

Canada and Latin America:
LAERDAL MEDICAL CANADA LTD.
151 Nashdene Rd., Unit #45
Toronto, ON, Canada, M1V 4C3
Tel. +1 (416) 298-9600,
Toll free 888/LAERDAL (523-7325)
ou en français (800) 567-9987
Fax +1 (416) 298-8016
E-mail: savelives@laerdal.ca

Denmark: LAERDAL DANMARK
Esplanaden 18, 1. sal
1263 København K
Tel. +45 80 333 112, Fax +45 80 333 555
E-mail: laerdal.denmark@laerdal.no

Eastern Europe: LAERDAL C.E. EUROPE
A-1060 Vienna, Mariahilferstr. 1d/1/3
Tel-Fax +43 15877140, Tel. +43 15815927
E-mail: mstam@aon.at

Far East: LAERDAL Singapore Pte Ltd
No 1 Marine Parade Central
#13-05, Parkway Centre, Singapore 449408
Tel. +65 63464259, Fax +65 63467523
E-mail: admin@laerdal.com.sg

Finland: LAERDAL OY
Vattuniemenranta 2, 00210 HELSINKI
Puhelin +358 (0)9-612 99 80,
Fax +358 (0)9-692 77 99
E-mail: laerdal.finland@laerdal.no

France: LAERDAL MEDICAL FRANCE
1 rue des Vergers - Bâtiment n° 5 - 69578 Limonest Cedex
Tel. +33 (0)4 72 52 02 52, Fax +33 (0)4 78 35 38 45
E-mail: laerdal.france@laerdal.no

Germany: LAERDAL MEDICAL DEUTSCHLAND GmbH
Am Loderfeld 56, 81249 München
Tel. +49 (0)89 / 864 954-0, Fax +49 (0)89 / 864 34 84
E-mail: info@laerdal.de

Italy: LAERDAL ITALIA s.r.l.
Via Piero Gobetti 52/2 int.Z. 40129 Bologna
Tel. +39 051-355587, Fax +39 051-355598
E-mail: laerdal.italy@laerdal.no

Japan: LAERDAL MEDICAL JAPAN K.K.
Shinjuku IS Bldg. 8F
2-13-12, Shinjuku, Shinjuku-ku, Tokyo 160-0022
Tel. 81-3-5919-2121, Fax: 81-3-5919-2122
E-mail: laerdal.japan@laerdal.co.jp

Malaysia: LAERDAL HOSPILINE Sdn Bhd
12 Jalan Titar 33/25, Section 33, Shah Alam Technology Park,
40400 Shah Alam, Selangor Darul Ehsan,
Tel +60 (3) 5122-7002, Fax +60 (3) 5122-5650
E-mail: customerservice@laerdal.com.my

New Zealand: LAERDAL NEW ZEALAND LTD
(GST Number 79-403-636)
PO Box 302664, North Harbour, Auckland 1330
Tel. +61 3 9564 4066, Toll free 0800 523 732
Fax +61 3 9563 3368, Toll free 0800 528 852
E-mail: customerservice@laerdal.co.nz

Portugal: LAERDAL ESPANA, Sucursal em Portugal
Rua da Bela Vista à Graca, no.31 A, Escritorio 9,
1170-054 Lisboa
Tel. (+351)-21-816-6567, Fax (+351)-21-816-6568,
Mob. (+351)-918765367

Spain: LAERDAL ESPANA, S.L.
Manuel Tovar 19, 28034 Madrid
Tel. (+34) 902-291110, Fax (+34) (91)-7291125
E-mail: laerdal.spain@laerdal.no

Sweden: LAERDAL MEDICAL AB
Box 2064, Paviļongvägen 5, 132 02 Saltjö-Boo
Tel. +46 (0)8-55614610, Fax +46 (0)8-55614619
E-mail: kontakt@laerdal.se

United Kingdom: LAERDAL MEDICAL LTD.
Laerdal House, Goodmead Road, Orpington,
Kent BR6 0HX
Tel. +44 (0)1689 876634, Fax +44 (0)1689 873800
E-mail: customerservice@laerdal.co.uk

USA:
LAERDAL MEDICAL CORPORATION
167 Myers Corners Road, PO. Box 1840
Wappingers Falls, New York 12590-8840
Tel. (800) 431-1055, +1 (845) 297-7770
Fax (800) 227-1143, +1 (845) 298-4545
E-mail: customerservice@laerdal.com

LAERDAL TEXAS
PO. Box 38, 226 FM 116 South
Gatesville, Texas 76528-0038
Tel. (800) 433-5539, (254) 865-7221
Fax (254) 865-8011
E-mail: customerservice@laerdal.com

This product is in compliance with the essential requirements of Council Directive 89 / 336 / EEC; EMC Directive.

Laerdal
helping save lives

Laerdal
helping save lives

Table of Contents

Warranty.....3
 Laerdal Recommends.....3
 Items Included.....3
 Skills Taught.....3
 Attachment of Blood Pressure Arm to VitalSim Unit.....3
 Attachment of Blood Pressure Cuff to Arm.....4
 Care and Maintenance4

Language Versions

Español 5
 Italiano 7
 Français..... 9
 Deutsch..... 11
 Nederlands 13
 Norsk..... 15
 Svenska 17
 Suomi..... 19
 中文..... 21
 日本語..... 23
 25

Warranty

Laerdal Medical warrants to the purchaser that its products are free from defects in material and workmanship for a period of one (1) year from the date of purchase by the original user. During the designated one (1) year period, Laerdal Medical will, upon receipt of a product found to be defective due to materials or workmanship from the purchaser and notification in writing of the defect, at its option repair or replace any parts found to be defective or the entire product. Warranty period does not renew with replacement or repair of original product.

Products found to be defective and notification of defects may also be sent to the authorized Laerdal Medical dealer from whom the product was purchased. All postage, shipping or handling charges shall be the sole responsibility of the purchaser.

Laerdal Medical is responsible for the effects of safety, reliability and performance of its product(s) only if:

- service, repair, readjustment or modification is carried out by Laerdal Medical or persons authorized by Laerdal Medical.
- the electrical installation of the room where the product is used complies with pertinent equipment requirements.
- the product is used in the proper manner in strict compliance with its Directions for Use.

Laerdal Medical shall not be liable under this warranty for incidental or consequential damages, or in the event any unauthorized repairs or modifications have been made or attempted, or when the product, or any part thereof, has been damaged by accident, misuse or abuse. This warranty does not cover batteries, fuses, normal wear and tear, excessive use, staining, discoloration or other cosmetic irregularity which does not impede or degrade product performance.

Some states in the USA do not allow the exclusion or limitation of incidental or consequential damages, so those limitations or exclusions may not apply to you.

There are no other express or implied warranties, whether of merchantability, fitness or purpose, or otherwise, on the product, its parts and accessories.

The VitalSim™ Blood Pressure Trainer is an accessory to the VitalSim, Vital Signs Simulator that allows for the measurement of noninvasive blood pressure, brachial and radial pulse checks.

The manufacturing quality of this simulator should provide many sessions of training when reasonable care and maintenance are practiced.

Laerdal Recommends
 For use with standard stethoscope.

Items Included:

- (1) VitalSim Blood Pressure Trainer*
- (1) Carry Case
- (1) Blood Pressure Cuff

*Must be used with VitalSim Vital Signs Simulator (sold separately).

Skills Taught:

- The measurement of Noninvasive Blood Pressure, either auscultated or palpated
- Radial and Brachial Pulse Checks

Blood Pressure Trainer Connections –

Attachment of Blood Pressure Arm to VitalSim Unit –

Attach serial cable and clear pneumatic tubing to back of VitalSim unit. (Photo 1)

Photo 1

Attachment of Blood Pressure Cuff to Arm –

- a. Place supplied Blood Pressure Cuff on arm.
- b. Attach clear tubing on cuff to clear tube attached to arm. (Photo 2)

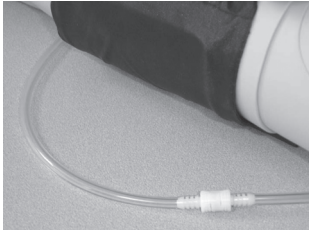
Photo 2

See VitalSim Directions for Use for complete Blood Pressure operating procedures.

Laerdal Recommends
Do not insert needles into blood pressure arm.

See VitalSim Directions for Use for additional connection and operating procedures.

Care and Maintenance –

1. Clean with mild soap and water; do not submerge trainer or parts in cleaning fluids or water. Apply a light coat of talcum powder to the face and chest skin to achieve skin-like feeling.
2. Use only on clean surface. Avoid felt tipped markers, ink pens, acetone, iodine or other staining products and avoid placing the manikin on newsprint or inked lines of any kind.
3. To ensure longevity, trainer should be cleaned after each training session and a general inspection should be conducted regularly.
4. Modules and all other parts should be drained and air-dried thoroughly before storage and disinfected when needed.
5. Articulating parts will benefit from a light application of talcum powder prior to training sessions.
6. Store properly between teaching sessions.

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